

ProCure 2

High-Speed Post Curing System Designed for Digital Dentistry

Quick Start Guide

Congratulations!

In this brief guide, you'll learn how to set up and use your ProCure 2 post-curing system.

Please follow the instructions carefully and visit our website for tips & tricks.

We're so excited to see what you make!


Training and Support

Contact our team of support technicians with questions.




Visit support.sprintray.com
Call 00800-70708000
Email support.eu@sprintray.com

1 In the Box

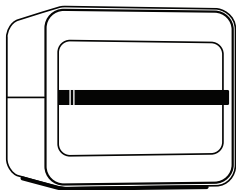

ProCure 2

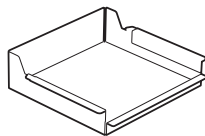

Curing Tray
Refer to section 5



Power Cable



USB Cable
With USB type C adapter




Onboarding Card

2 Where to Set Up


Situate ProCure 2 in a well-ventilated area on a flat, level surface. Keep it away from extreme temperatures, windows, and direct sunlight.



Hot air exhausts through ventilation located on the back and bottom of ProCure 2. Do not obstruct.

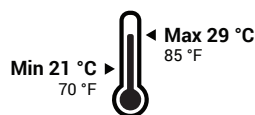

Recommended
Operating Temperature

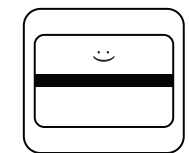

Level
Surface





Uneven Surface
Warm Environment

3 Plug in ProCure 2

Use the provided power cable to plug ProCure 2 into a surge protector or uninterrupted power supply. Turn it on by flipping the power switch located on the back of the unit.


To perform a reset, use the power switch or disconnect ProCure 2 from its power source.

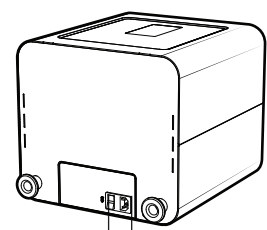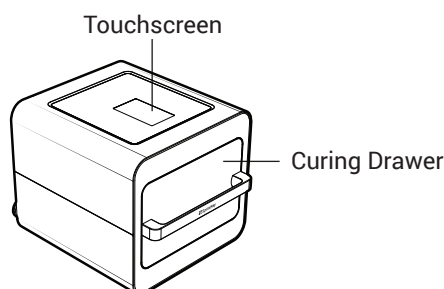

Power Switch — Power Port

4 Connect to a Wireless Network

Press the wireless connection icon using the touchscreen and follow the steps to connect to your network. Firewall or antivirus software may prevent ProCure 2 from establishing a reliable connection. Consult SprintRay customer support if you need help connecting to a network.

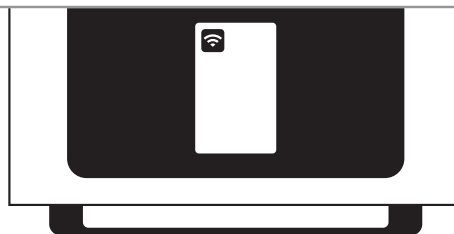


5 Install Curing Tray

Place Tray Inside Curing Drawer

Open the UV drawer by pulling on the handle.


Set the Curing Tray in the drawer so that it rests inside the frame. Close the drawer slowly to ensure the tray is properly in place.

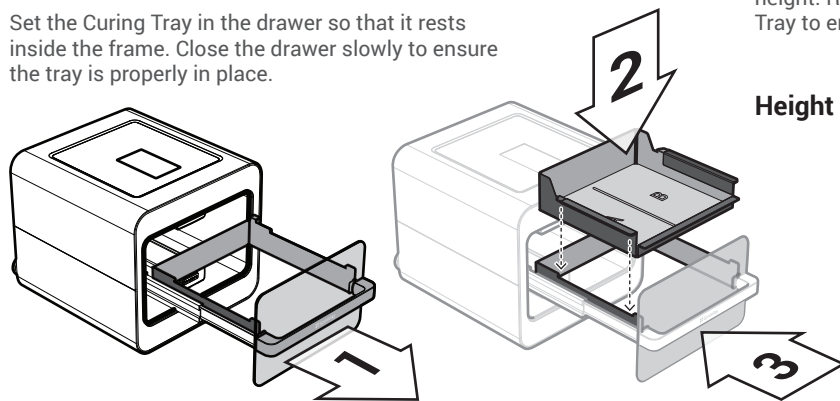

Height Limit is 75 mm

Models inserted into ProCure 2 cannot exceed 75 mm in height. Refer to the height limit indicator on the Curing Tray to ensure optimal performance.

Height Limit

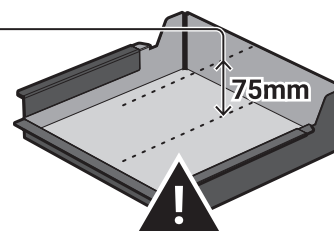

Do not try to close the curing drawer if models exceed the height limit; this may permanently damage ProCure 2.

6 Using ProCure 2

How to Operate

Pull on the handle to open the curing drawer. Place models on the curing tray and close the drawer. Select desired material from the touchscreen to begin the process.

Understanding Zone Selection

ProCure 2 can selectively cure the model tray to reduce time when fewer models are being processed. You can select which zones to cure from the touchscreen.

Zone A Only

If all your models fit into **Zone A**, select this to save time.




vs.



Zones A and B

If you are processing a large number of models, select **Zone A and B**.

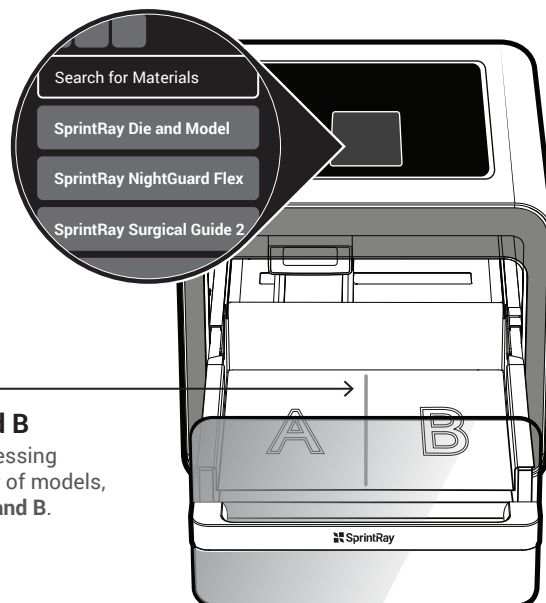

WARNING

ProCure 2 may become warm to the touch during post-curing. Do not obstruct the ventilation located on the rear and bottom of the unit.


7 Maintenance

Software Updates

ProCure 2 will periodically receive updates over the air via wireless network. Ensure ProCure 2 is connected to the internet to receive updates.

Troubleshooting

Please call customer support to perform advanced troubleshooting. A USB cable with a type C adapter is provided to connect ProCure 2 to your computer if needed.

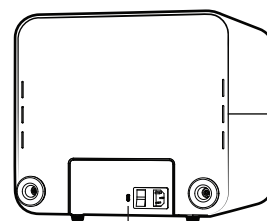

USB Type C Port
For Manual Updates