
## **Install Curing Tray Inside ProCure 2**


- Step 1 Use the handle to pull open the curing drawer.
- **Step 2** Lower the curing tray into the frame as indicated in the image below.
- Step 3 Close the drawer cautiously, ensuring the tray is properly seated.

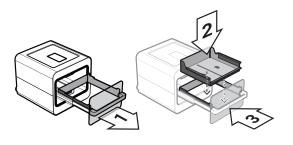

## **Height Limit Warning**

#### Height Limit is 3" (75mm)

Models cannot exceed 3" (75mm) in height.

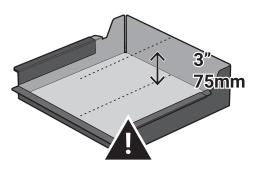

Do not force the curing drawer closed if the models exceed the height limit. This may permanently damage ProCure2.

## **Curing Zones, Explained**

ProCure 2 offers you the choice of two curing zones depending on the volume of models being post-processed.

# Zone A only 'A' Box Only = Small Batches

Zone A is the fastest way to cure, but will only cure models within the bounds of the 'A' box on the curing tray.



#### Zone A and B

Full Tray = Large Batch

Zone A & B allows you to cure the entire tray area, but will increase the program duration by around 50%.

